

# BT-Cath<sup>®</sup>


Balloon Tamponade Catheter for  
Postpartum Hemorrhage



*Simple & Timely When it's Needed Most.*



Catalog No. BTC-100

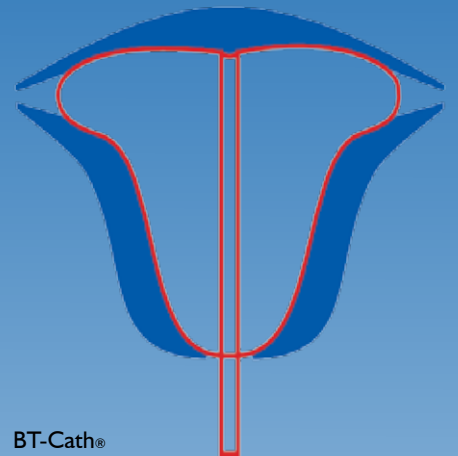

**BT-Cath's soft silicone balloon material**  
contours to the internal uterine shape.

The intrauterine drainage port is flush with the top of the  
balloon allowing **placement near the fundus.**

Several design components  
facilitate **timely balloon inflation.**

UTAH MEDICAL  
PRODUCTS INC.



Call customer service  
at **800-533-4984** or  
access  
**[www.utahmed.com](http://www.utahmed.com)**  
for more information.



U.S. Patent 8,123,773

PN: 58304 REV: 070218

# BT-Cath<sup>®</sup>

with EasyFill™ Inflation System (Catalog No. BTC-ESY)


- Single-person, simplified operation.
- Allows inflation directly from the fluid bag.
- Creates a closed system.

**1. Spike** a 500 mL bag of fluid.

**2. Prime the tubing** by attaching the syringe and drawing fluid through the tubing.

**3. Inflate** by connecting EasyFill™ to BT-Cath and pushing fluid into the balloon.

UTAH MEDICAL  
PRODUCTS INC.

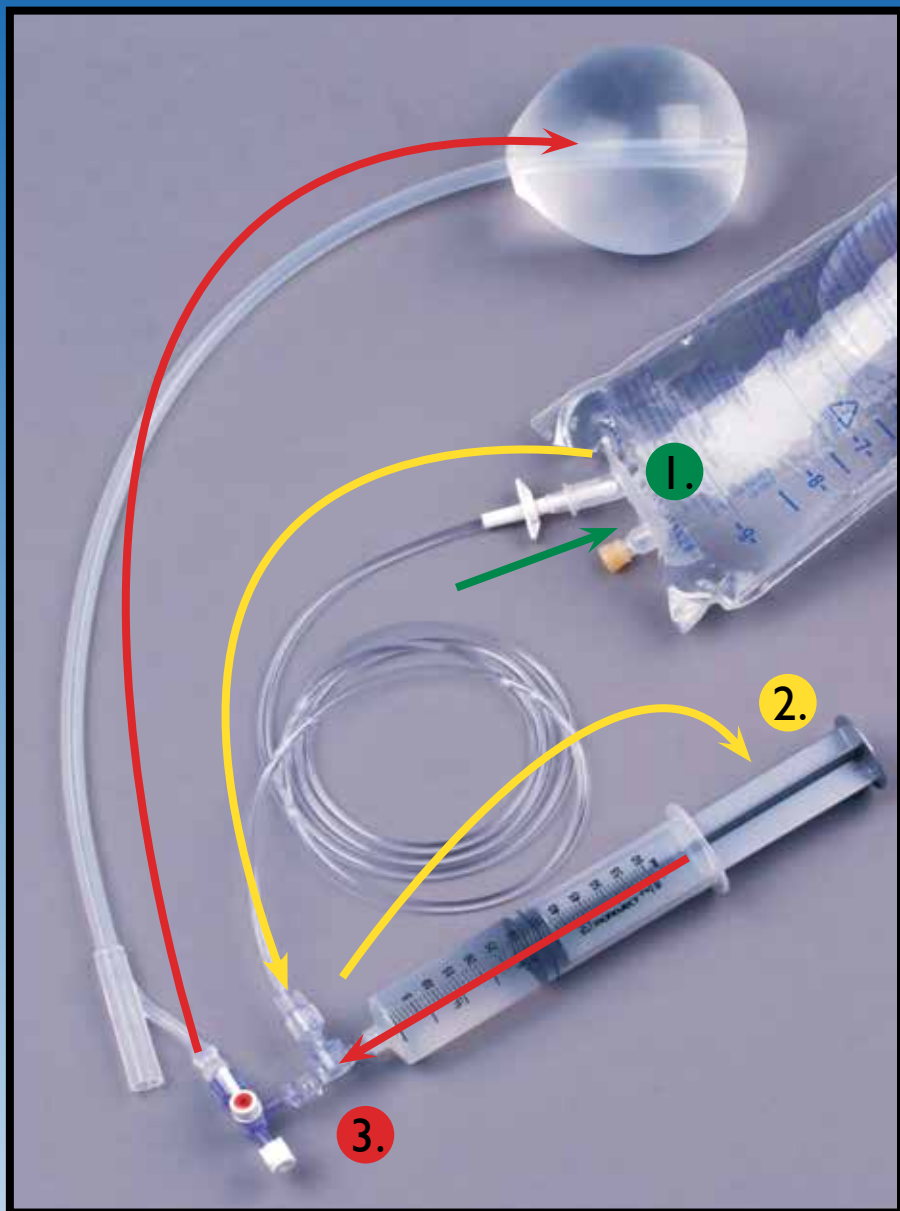

## Catalog Numbers

BTC-100

BTC-ESY

## Description

BT-Cath with bag spike and 2 syringes

BT-Cath with EasyFill inflation system

## Packaging

2/box

2/box

## Customer Service

US: 1-800-533-4984

UK: (01794) 525100